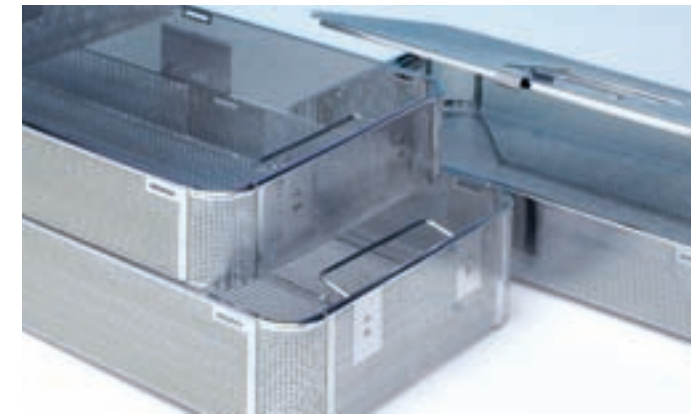
# CONTAINER SYSTEM

**C.B.M. s.r.l. Medical Equipment**  
26038 Torre Picanardi (Cremona) Italy - Via Castello, 10/A  
Tel. ++39 0375 394095 - Fax ++39 0375 394098  
E-mail: info@cbm-srl.com - www.cbm-srl.com

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The widest and most complete product line on the market.

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## The solution for every need

The C.B.M. container system represents the best choice for modern sterilizing centres. The flexible nature of the extremely large range of containers available offers the operators an answer for every requirement in matters of packaging and transporting surgical apparatus. C.B.M. is, in fact, the only producer with the capability of proposing such an extensive assortment of dimensions, materials and different filtering systems, as well as a wide selection of accessories, conforming to the quality requirements of European standard regulations.

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## Material used

C.B.M. uses materials of recognized trustworthiness for the production of its containers, conforming to the requirements of the regulations in force.

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## Aluminium containers

The containers manufactured by C.B.M. are composed of a Magnesium-Aluminium alloy. The aluminium requires a special process in order to resist the aggression of corrosive agents. This process, known as anodizing, makes for the creation of a surface coating that renders the aluminium resistant to attack from water steam. By means of anodizing, it is possible to obtain covers in different colours. In this way, it becomes possible to differentiate the destination departments of the containers on the basis of the colour code for the covers. As well as the use of the classical anodizing process, C.B.M. manufactures its own containers with a process called "hard anodizing". This treatment **boosts the mechanical surface properties** of the aluminium, giving the container a brownish colour.

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## Containers in stainless steel

The C.B.M. stainless steel containers can be considered to be the **best choice** for a modern sterilizing centre. The material used is AISI304, an austenitic stainless steel with **exceptional mechanical resistance and highly resistant to corrosion**. The containers in stainless steel are naturally protected against aggression by chemical components without any need for further surface treatment. Consequently, scratches and dents do not impair the resistance to corrosion, assuring a **prolonged working life**. AISI304 possesses **mechanical properties** that are far superior to aluminium, and this allows for the manufacture of a container with a much thinner wall. It is, therefore, a **mistake to suppose that a container in steel is heavier** than one made from aluminium. Comparing two containers manufactured in both these materials and filled with the same quantity of product it is found that the difference in weight is less than 4%. Furthermore, the **transmission of heat is identical**. Considering the thickness of the wall of a steel container and of an aluminium container, it can be shown that each offers the same resistance to heat transmission towards the load being sterilized.

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### Loading information - rigid containers for steam sterilization

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(Norma DIN 58953-9 / Sept. 2000)

## Maintaining sterile conditions

A container can maintain sterile conditions for up to **6 months** when the following recommendations are complied with:

- the storage site must be dust-free;
- the storage site must have controlled pressure, temperature and humidity;
- the storage site surfaces must be smooth and easy to disinfect;
- the first shelf of shelving must be at least 30 cm. above the floor level;
- air flows must be controlled;
- all operating, maintenance and cleaning instructions furnished by the manufacturer of the container must be complied with.

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## Guarantee and working life

• Parts of containers not subject to wear are guaranteed for 1 year against manufacturing defects or breakdowns in components that are not due to poor or improper use by the user.

• Steel parts of the carrying handles are guaranteed for the life of the product

• Container life is 2000 cycles or 5 years, if the number of cycles is less than 2000.

• Cover and filter seals must be replaced every 500 cycles or every year if the number of cycles is less than 500.

• The paper filters must be replaced every cycle.

• The cotton filters must be replaced every 70 cycles.

• Synthetic filters must be replaced every 200 cycles.

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## Reference standards

- **EEC Directive 93/42**, June 14, 1993 - Medical Devices;
- **ISO EN 9001:2000** - Quality systems - Quality assurance criteria;
- **ISO EN 13485** Medical devices - Quality management systems.
- **EN 556** - Sterilization of Medical Devices - Requirements for medical devices that carry the "sterile" message;
- **EN 554** -Sterilization of Medical Devices - Method for certification and systematic inspection of steam sterilization;
- **EN 285** -Sterilization of Medical Devices - Large steam sterilizers;
- **EN 868-1** -Systems and materials for medical devices that must be sterilized - General requirements;
- **EN 868-2** -Systems and materials for medical devices that must be sterilized - Sterilization pack - Requirements and test methods;
- **EN 868-3** -Systems and materials for medical devices that must be sterilized - Paper for use and manufacture of paper containers - Requirements and test methods;
- **EN 868-8** -Systems and materials for medical devices that must be sterilized - Reusable containers for steam sterilizers conforming to **EN 285** Standards;
- **EN 10088-1** -Stainless steels;
- **EN 867-1** -Non-biological systems for use in sterilizers - General requirements;
- **EN 867-2** -Non-biological systems for use in sterilizers - Process indicators.

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## "Filter" model container

C.B.M. Medical Equipment has pleasure in presenting the latest generation of containers produced through the evolution of the celebrated "Filter" model. The containers in austenitic stainless steel and in anodized aluminium have been fitted with a new filter lock made of technopolymer. The new system for fixing the filters is actually manufactured from polysulfone resin, a thermo-plastic material that known to be efficient for steam sterilizing. **Polysulfone is characterized by an elevated resistance to high temperatures and is outstanding for excellent properties of resistance to mechanical stress and to chemical agents.** The use of polysulfone, together with the stainless steel construction of the container, further guarantees a **high resistance to the new processes for cleaning and disinfecting by means of alkaline and oxidizing agents.** The new filter lock has been designed to facilitate replacement of the containers' filters. The system for closing ensures **rapidity and practical ease.** The filters are applied in a safe manner and the silicone gasket of the filter lock ensures that air and steam can enter only through the filter. In addition, the gasket is not glued in place and can be removed easily by the end-user for cleaning and maintenance operations. Thanks to the new geometrical design of the filtering system it is also possible to optimize handling of material subject to wear. The filters are, in fact, universal, and can be applied indiscriminately to containers of different dimensions.

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
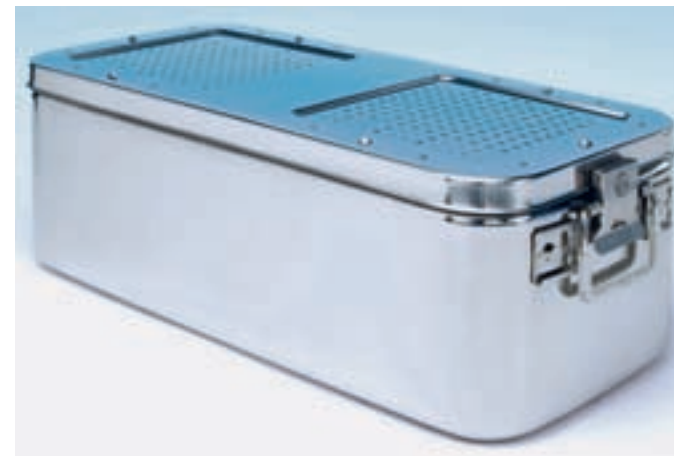
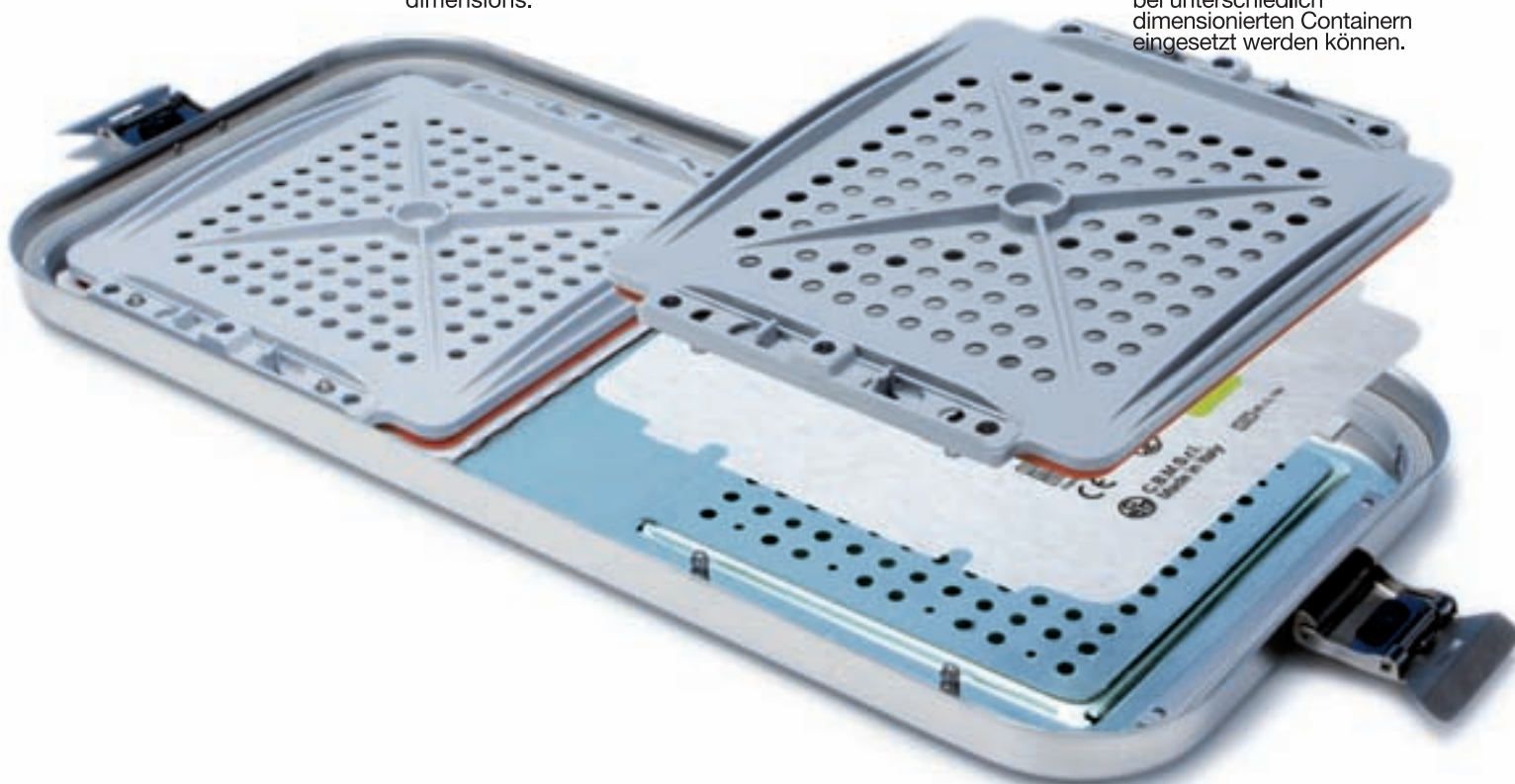
In case of order please specify the number of the required filters: SF, 1F, 2F, and if the lid is to be with or without hinges: C/C or S/C.

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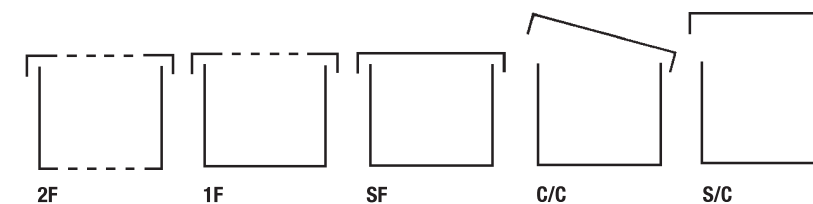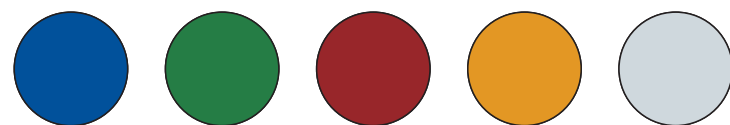
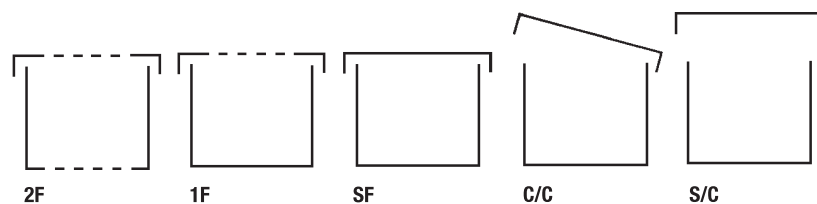
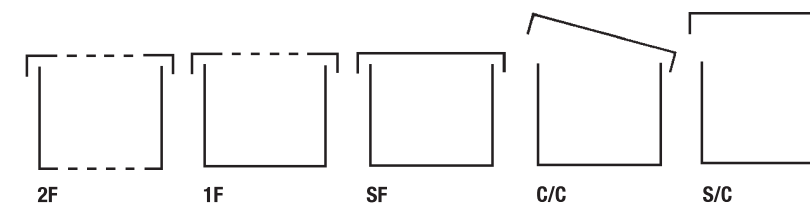
### “Protected Filter” model container

Besides the characteristics of the filtering system, which are the same as the Filter Model Container, the Protected Filter Model Container is provided with a double lid. The inlet of steam and air occurs horizontally on the lid sides, through a tortuous run which prevents dust from depositing on the filter. The vent position does not allow pointed and sharp objects to get to the filter, damaging it and affecting the tightness and sterility of the load. The double lid avoids that condensate from the containers piled up on it, comes into contact with the filter which, getting wet, could lose its characteristics as microbic barrier.

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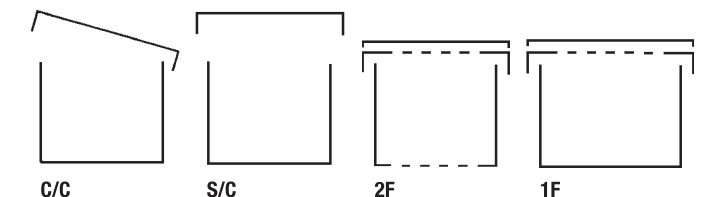
In case of order please specify the number of the required filters: 1F, 2F, and if the lid is to be with or without hinges: C/C or S/C.

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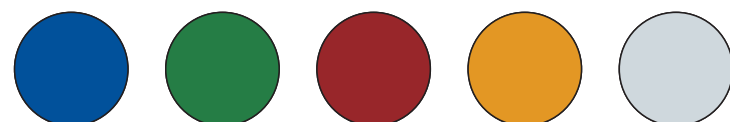
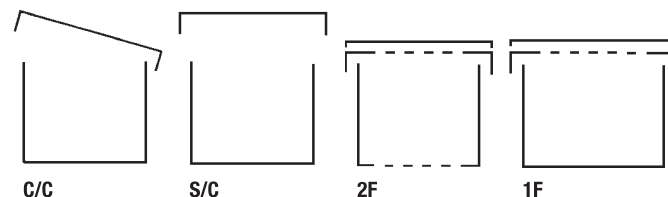
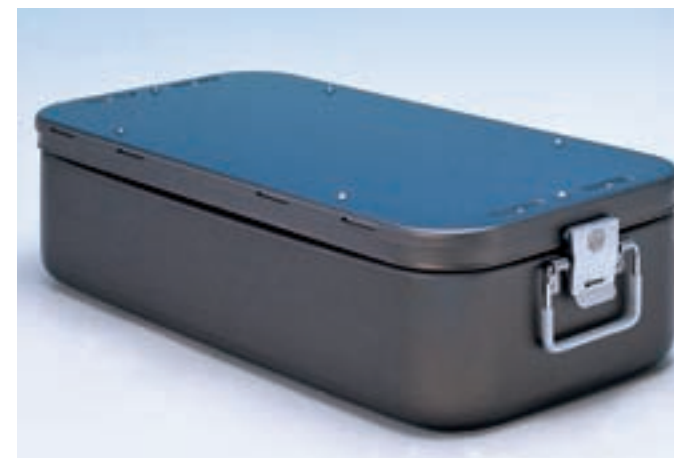
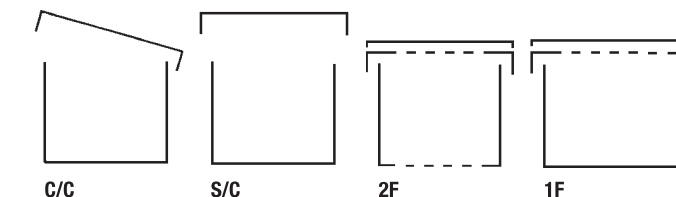
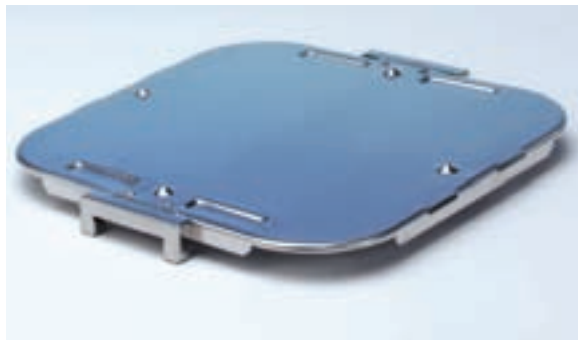
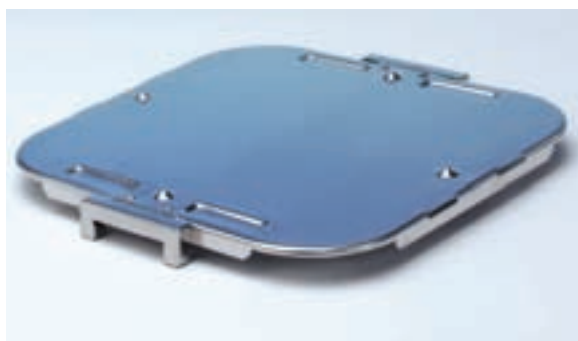
## “Protected Valves” model container

The Protected Valves Model Container is a mechanical filtering system requiring no maintenance. Its correct functioning may be controlled by inserting a tip in the suitable hole in the centre of the valve bottom. The valve’s three working positions during and after sterilization are shown in the drawing.

**1** Before sterilization the valve is in state of rest.

**2** During the vacuum stage depression makes the valve open completely, so that air escapes from the container.

**3** Afterwards the valve closes, and the second valve opens; in this stage steam enters and the sterilization stage starts.

**1** At the end of the cycle the valve closes air-tight in order to maintain the container load sterile.

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## “Side Filter” model container

The C.B.M. patent concerned with the lateral filter system further improves the excellent capacities of sealing, penetration of steam and filtration of the air for our range of rigid containers for steam sterilizing, being offered as an efficient and economic solution for sterilizing units. The principal novelties of this system lie in the vertical position and in the reduced dimensions of the filters. Summing up below the reasons for which these characteristics make the system more efficient:

- The vertical position of the filters overcomes the penetration problems of the steam that arise from a faulty stacking.
- The inlet for the steam always lies at the same distance from the centre of the load;
- Dust, dirt and condensed water do not accumulate on the filters protecting the sterility of the load during its transportation and storage;
- The dimension of the filter is optimized according to the height of the container, and to a low container there is a correspondingly low filter. In economic terms this is translated into a lowering of the cost of management for the consumables.

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## The Mini containers is the answer for small- scale operations.

C.B.M., exploiting the experience it has gained in the production of Containers, has developed a series of small containers for sterilization. The C.B.M. series of "Mini-Containers" allows for the preparation of single-patient surgical sets for minor operations. The "Mini-Containers" possess all the characteristics of robustness and flexibility in use of the regular C.B.M. containers. Indeed, they are manufactured using high-quality materials and in conformity with the European standard regulations.

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The new basket design with round corners has some features that distinguish this product out in the market. They are used in sterilisation practice together with the containers or alone with paper wrapping and, in combination with the lid, as support for instrument washing. The perforated sheet basket are entirely made in stainless steel grade AISI 304. The baskets are electrically polished. This process avoids the colour changing due to the high temperature of the sterilisation and avoids the formation of water stains. The new round corners baskets:

- do not create any problem of scratches while using paper wrapping;
- increase the usable space of the basket;
- in order to avoid that sharpened instruments pass through the sides, the C.B.M. baskets have different perforation sizes between in the bottom (4x4 mm) and the sides (2x2 mm).

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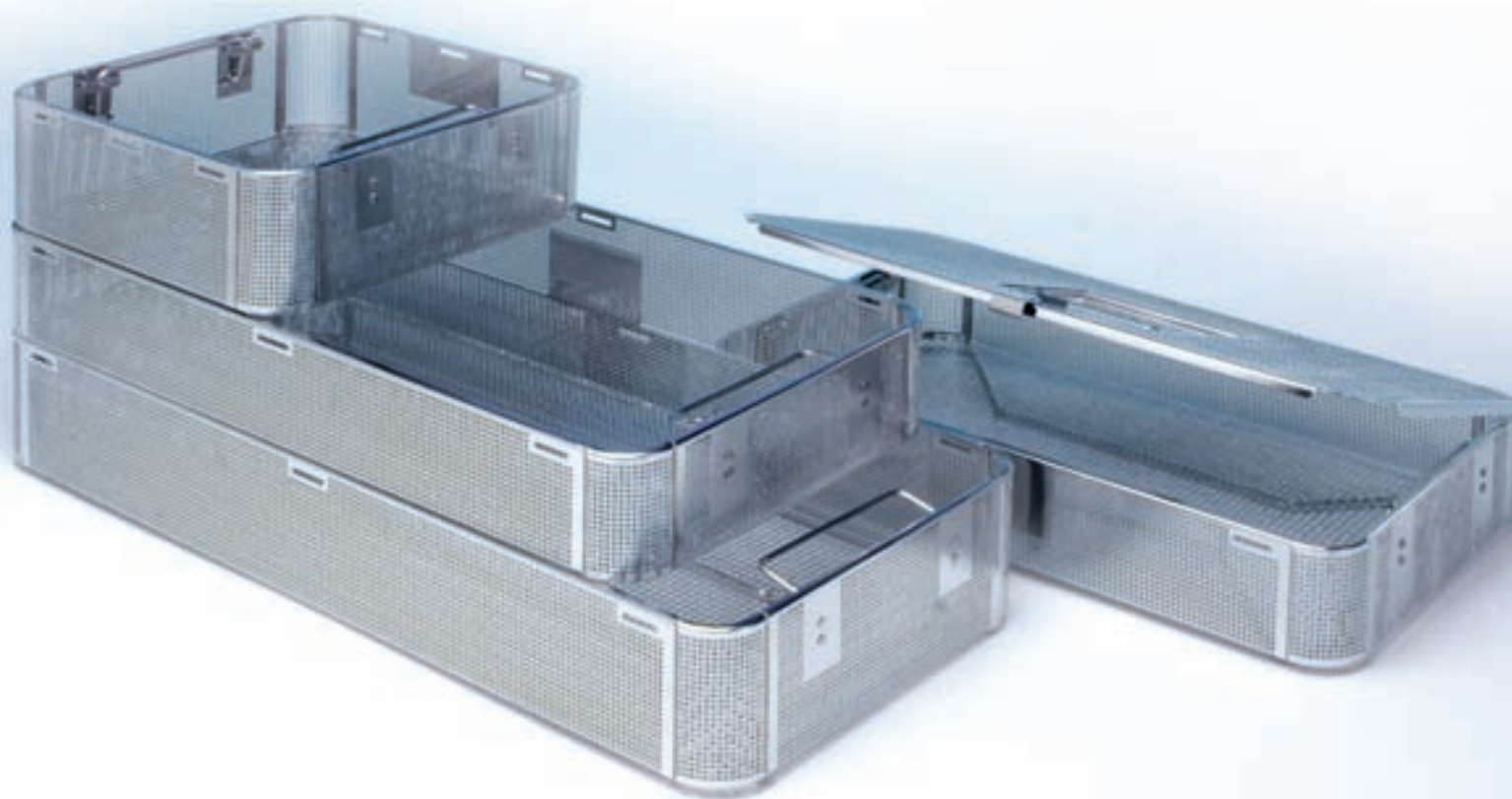
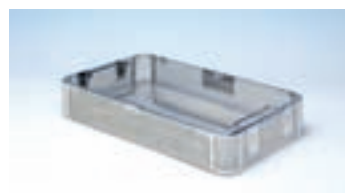
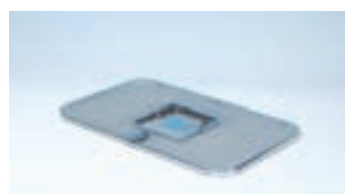
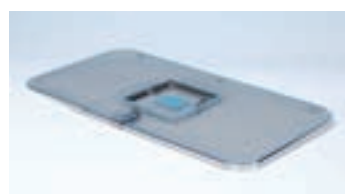
The range of standard size is wide and on demand it is possible to have personalized sizes. It is also possible evaluate the possibility of a customized product.

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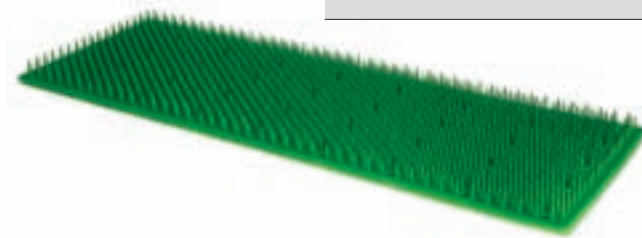
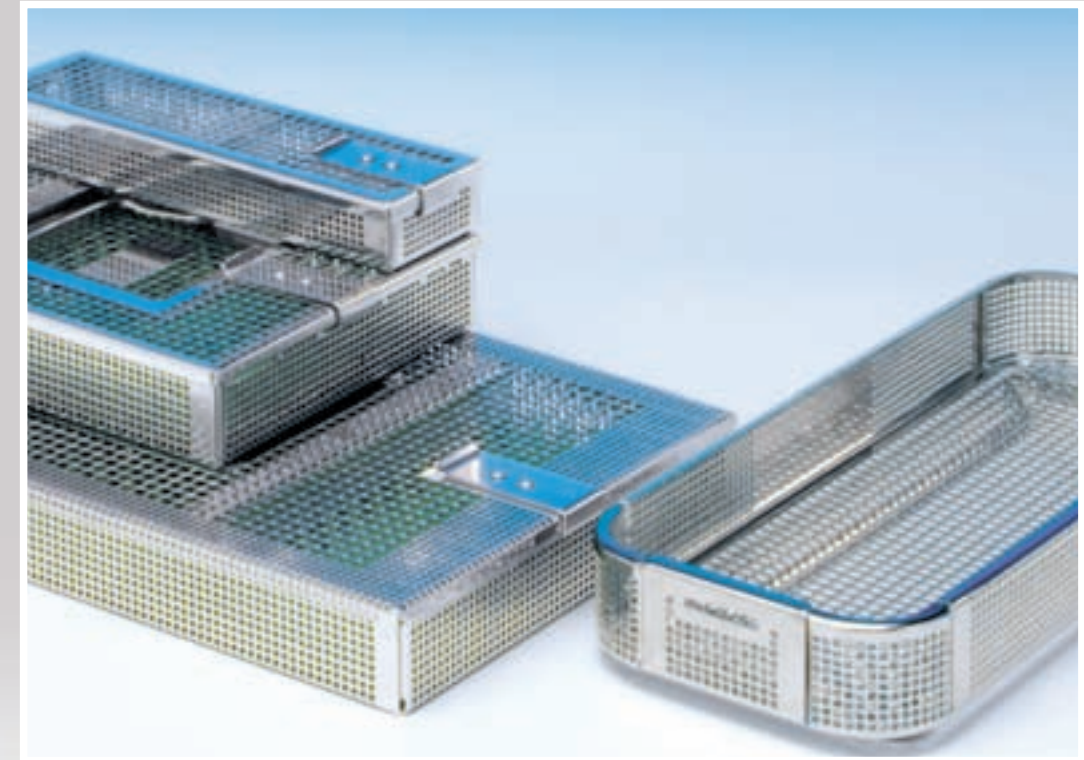
## Mini-baskets

Employing its great experience gained in the medical field, C.B.M. has developed a new range of tray baskets for sterilizing, cleaning and transporting small instruments (e.g. instruments used in the dental department, in micro-surgery in ophthalmology,...). These baskets are produced in the highest quality stainless steel (AISI 304).

The characteristic design of the C.B.M. tray baskets, thanks to the different dimensions of the pores, (4x4 mm on the cover and on the body, 2x2 on the sides) is a guarantee of safety in the treatment of small sharp instruments. The dimensions of the holes so envisaged avoids the danger of lateral emissions of sharp points and at the same time allows for the correct penetration by steam in the course of sterilization or of detergents used during the washing.

The baskets are electrolytically polished. This process avoids colour changes caused by the high temperatures in the course of sterilizing and inhibits the formation of water spots.

The baskets are fitted with a fixing system that prohibits shifting of the instruments. When the basket is closed, the instruments are delicately clasped in a gentle and safe manner (on the surface of a perforated mat in silicone and on the cover by silicon swallow-tail flaps of different dimensions).

All the baskets can be ordered with personalized fixing systems or, if required, without. It is possible to appraise the design of a personalized basket.

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